

Prevention and treatment of infections

VERADERMTM

WOUND WASH SALINE SPRAY

Action:

- 0.9% saline is generally considered compatible with living tissues
- Exert a mechanical cleansing action for tissues or wounds
- Normal saline helps in moisturizing the wound
- Wounds need moisture in order to heal properly. If a wound is too dry it simply will not heal. In some instances, the normal saline is used in a wet-to-dry or damp-to-dry dressing to moisturize the wound
- Just point and spray
- Unique 360 degrees dispensing allows can to be held in any position.



Facts for saline spray

- Isotonic solution
- Do not impede normal healing
- Do not cause allergy or
- Do not alter the normal bacterial flora of the skin
- Also saline remains the cleansing fluid of choice in the hospital setting

100 ml
packing

VERADERMTM

WOUND WASH SALINE SPRAY

Indications

- Each of these solutions is indicated for all general irrigation, washing, rinsing and dilution purposes
- Minor cuts or wounds or lacerations
- Ulcers
- Burns
- As an First Aid measure

Directions

- Before use, expel a short stream of Wound Wash Saline to clear nozzle.
- To dispense, aim nozzle and press. Apply spray to affected area.
- Flush area with Wound Wash Saline. Adjust distance as necessary.
- May be applied as often as needed or as directed by a healthcare professional.

Warnings

- Contents under pressure.
- Do not puncture or incinerate.
- Keep out of reach of children.
- Do not store at temperatures above 120 degrees F.
- Avoid excessive heat.
- For external use only. Not for injection.



Sutton Healthcare International
1557 Garland Cres Pickering, On CA LIV 7B2, Canada
www.suttonhealth.com